ONE TOUCH[®] Delica[®]

Lancing Device

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Autopiqueur

The OneTouch[®] Delica[®] Lancing Device is only compatible with OneTouch[®] Delica[®] Lancets. When purchasing replacement lancets, be sure to ask for OneTouch[®] Delica[®] Lancets.

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The OneTouch[®] Delica[®] Lancing Device provides a safe, easy way to obtain a drop of capillary blood.

Your OneTouch[®] Delica[®] Lancing System does not include the materials needed to perform Alternate Site Testing (AST). The OneTouch[®] Delica[®] Lancing System should not be used to test on the forearm or palm with the OneTouch[®] Ultra[®] Family of Meters, OneTouch Verio[®] Family of Meters, or the OneTouch Ping[®] Meter Remote. If you want to test on your forearm or palm, please contact LifeScan Customer Service (United States) at 1 800 227-8862 (English) 1 800 381-7226 (Spanish) or OneTouch[®] Customer Care Line (Canada) at 1 800 663-5521 (English / French) for the necessary materials and AST instructions.

The device has seven puncture depth settings, numbered 1 through 7. The setting you choose must be deep enough to get a round drop of blood large enough to get an accurate result with your meter (see your User Guide that came with your system for information on the proper blood sample size.)

NOTE: The OneTouch[®] Delica[®] Lancing Device uses ONLY OneTouch[®] Delica[®] Lancets. Use a new, sterile lancet each time you test. Lancet reuse can dull or bend the tip of the lancet, causing damage to your skin, scarring and greater pain.

CAUTION: Keep the lancing device away from young children. The lancing device, cap, lancets and lancet protective covers are choking hazards.

CAUTION: To reduce the chance of infection:

- Make sure to wash the puncture site with soap and water before sampling.
- Never share a lancet or a lancing device with anyone.
- Always use a new, sterile lancet — lancets are for single use only.
- Keep your lancing device clean.

Choose a different puncture site each time you test. Repeated punctures in the same spot may cause soreness and calluses.


Before testing, wash your hands and the sample site with warm, soapy water. Rinse and dry completely.

Preparing your lancing device

1 Remove the lancing device cap

Remove the cap by turning it counterclockwise and then pulling it straight off of the device.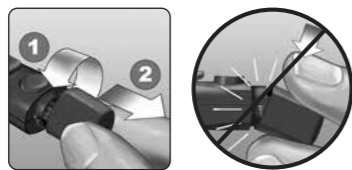

2 Insert a sterile lancet into the lancing device

Align the lancet as shown here, so that the lancet fits into the lancet holder. Push the lancet into the device until it snaps into place and is fully seated in the holder.


Twist the protective cover one full turn until it separates from the lancet. **Save the protective cover for lancet removal and disposal.** See *Removing and disposing of the used lancet.*

3 Replace the lancing device cap

Place the cap back onto the device; turn the cap clockwise to secure it.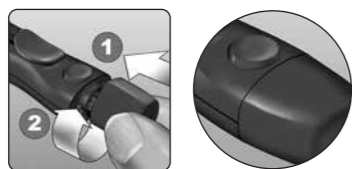

Do Not overtighten the cap.

4 Adjust the depth setting

The lancing device has seven puncture depth settings, numbered 1 through 7. Smaller numbers are for a shallower puncture and the larger numbers are for a deeper puncture. Shallower punctures work for children and most adults. Deeper punctures work well for people with thick or callused skin. Turn the depth wheel to choose the setting.

NOTE: A shallower fingertip puncture may be less painful. Try a shallower setting first and increase the depth until you find the one deep enough to get a blood sample of the proper size (see your User Guide).

5 Cock the lancing device

Slide the cocking control back until it clicks. If it does not click, it may already have been cocked when you inserted the lancet.

Sampling blood from the fingertip

1 Puncture your finger

Hold the lancing device firmly against the side of your finger. Press the release button. Remove the lancing device from your finger.

2 Get a round drop of blood

Gently squeeze and/or massage your fingertip until a round drop of blood forms on your fingertip.

3 Apply the sample

Apply the blood sample. Follow the instructions in your User Guide for proper sample application.

If the blood smears or runs, **Do Not** use that sample. Dry the area and gently squeeze another drop of blood or puncture a new site.

Removing and disposing of the used lancet

NOTE: This lancing device has an ejection feature, so you do not have to pull out the used lancet.


1 Remove the lancing device cap

Remove the cap by turning it counterclockwise and then pulling it straight off of the device.

2 Cover the exposed lancet tip

Before removing the lancet, place the lancet protective cover on a hard surface then push the lancet tip into the cupped side of the cover.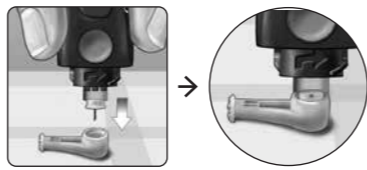


3 Eject the lancet

Slide the ejection control forward until the lancet comes out of the lancing device. Return the ejection control to its back position. If the lancet fails to eject properly, cock the device again and then slide the ejection control forward until the lancet comes out.

4 Replace the lancing device cap

Place the cap back onto the device; turn the cap clockwise to secure it.

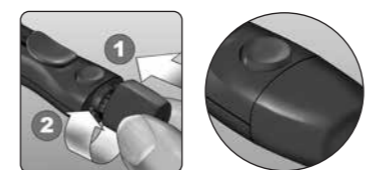Do Not overtighten the cap.

It is important to use a new lancet each time you obtain a blood sample. This will help prevent infection and sore fingertips.

Discard the used lancet carefully after each use to avoid unintended lancet stick injuries. Used lancets may be considered biohazardous waste in your area. Be sure to follow your health care professional's recommendations or local regulations for proper disposal.

Cleaning your lancing device and cap

Refer to the User Guide that came with your meter for cleaning and disinfecting. If there are no instructions provided for this lancing device and cap in the User Guide, follow the instructions below.

To clean these items, wipe them with a soft cloth dampened with water and mild detergent. **Do Not** immerse the lancing device in any liquid.

To disinfect these items, prepare a solution of one part household bleach to nine parts water. Wipe the lancing device with a soft cloth dampened with this solution. Immerse the **cap only** in this solution for 30 minutes. After disinfecting, rinse briefly with water and allow both to air dry.*

*Follow manufacturer's instructions for handling and storage of bleach.

OUR COMMITMENT TO YOU:

As your partner in diabetes care, we offer valuable diabetes-related knowledge, tools and special offers online. Should you need assistance, we welcome you to contact us.

LifeScan Customer Service (United States)

7 days a week, 8 a.m. - 8 p.m. Eastern Time.
1 800 227-8862 (English)
1 800 381-7226 (Español)
www.OneTouch.com

OneTouch[®] Customer Care Line (Canada)

Monday-Friday 9am-8pm Eastern Time, 6am-5pm Pacific Time
Canada: 1 800 663-5521 (English / French)
www.OneTouch.ca

For assistance outside of these hours, please contact your healthcare professional.

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© 2012-2018 LifeScan, Inc.
LifeScan Europe GmbH